


MK-IS01

Drip Stand

User Manual

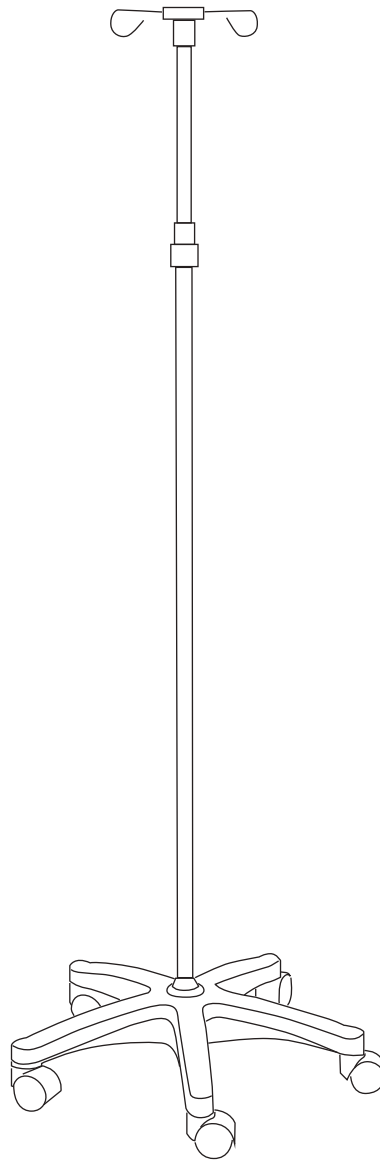

1.Scope of application

This IV stand was built to have transfusion bags etc. attached to it for intravenous drips.

2.Parameter & Function

Height adjust 140-240cm


Castor Dia 50mm

3.Assembly Drawing of Stool



4. Assembly Steps:

Please check the assembly picture .

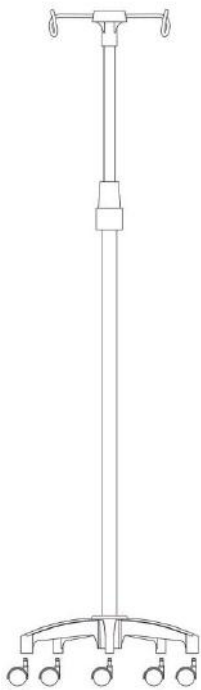

1. The one piece pole has two threaded ends connected with a locking collar.
2. Lay the pole on the ground. Place the base with the preassembled casters on its side next to the pole. Secure the longer bottom portion of the pole through the hole in the base. Securely tighten the threaded nut.
3. Screw the I.V. hook unit onto the shorter threaded top portion of the pole.
4. Adjust the height of the pole by loosening and then tightening the collar firmly by hand.
5. Periodically check that the thread bolt is properly tightened in the bottom of the base and that the nuts on the casters are secure. Ensure the I.V. Hook is tightly secured.
6. Cleaning of the I.V. Stand should be in accordance with the facility's maintenance policies.

5. General Warning

- DO NOT use this product without first reading and understanding the instructions contained in this booklet.
- Caution should be taken if attaching multiple pumps to the I.V. Stand. Pumps should be hung so that they counter balance each other.
- All wheels must be in contact with the floor at ALL TIMES.
- DO NOT use the I.V. Stand while walking backwards, down gradients, or climbing stairs, curbs or to go over obstacles. Serious risk of fall or injury may occur.
- DO NOT use in MRI areas.
- DO NOT use the I.V. Stand for support or as a walking aid. Do not attempt to support your weight or use as a stand assist.
- DO NOT hang or support any other objects that are not intended to be used with the IV pole.
- Use care when moving the I.V. Stand with a patient. Avoid wet or slick surfaces, wires, cables, or tubes and any uneven surfaces that may cause the I.V. Stand to tip or fall.
- DO NOT lift the pole by the top half of the pole.
- Extreme caution should be taken when the IV pole is near the bed or any other furniture

CONTACT US



Youtube



Web

Pls scan the QR code for more info

-  + (86) 139 6223 1292
-  info@medik-medical.com
info@mediksurgical.com
-  www.medik-medical.com
www.mediksurgical.com